# HeatMask+

## ■ HeatMask+ ))))

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## ■ HeatMask+ ))))

*The **HeatMask+** electrical heating mask is the essential device for face cares: it is used as well to clean deeply the skin as to help penetration of cosmetics active principles ; it restores tonus and vitality to every type of skin.*

*The **HeatMask+** is the alone 2 section heating mask in the world: 1 small section for upper part of face, 1 large section for lower part of face, including chin and neck down to shoulders; large section has 2 temperature settings: half and full power.*

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The HeatMask+ simultaneously acts through 3 ways:

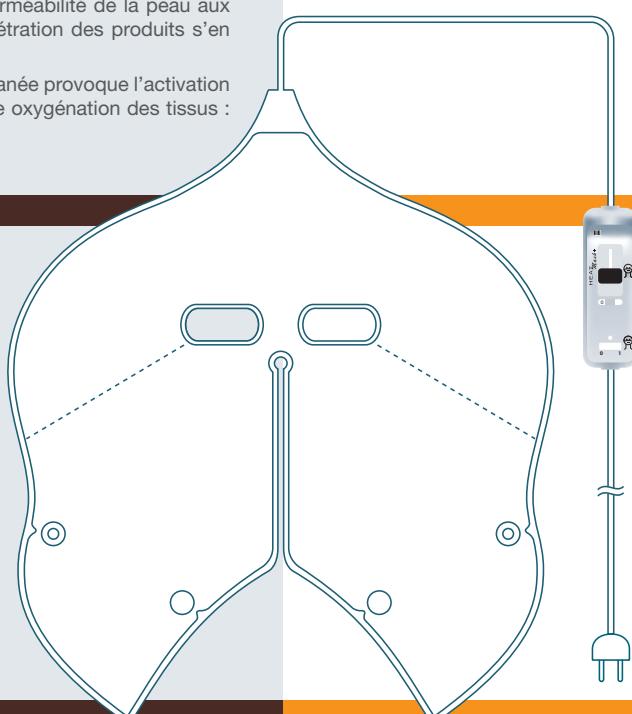
- it makes easier the elimination of toxins off the skin,
- it optimizes the active constituents of cosmetics,
- it activates blood micro-circulation.

## ■ PHYSICAL PRINCIPLE

The HeatMask+ 2 section heating mask emits FIR (Far Infra Red) rays of which wave lengths are from 7µm to 14µm ; since human body emits from 6 to 20µm FIR rays, FIR rays emitted by the HeatMask+ heating mask are called bio-compatible: as efficient as harmless to warm human body.

## ■ PHYSIOLOGICAL PRINCIPLES

- Detoxification: the "sauna" effect of the HeatMask+ enables the skin to remove its toxins off the skin: the skin is deeply cleaned.
- Penetration: the heat causes the dilatation of the pores, which results in improved skin permeability.
- Oxygenation: the skin temperature rise causes the activation of blood micro-circulation, and therefore tissue oxygenation: the skin then breathes deeply.

## ■ HYGIENE

HeatMask+ heating mask is built-in hospital care quality treated: this is a powerful anti-microbial and anti-fungal agent which is integrated into the mass of the outside cover.

Unlike current surface treatments, whose action is limited in time, this treatment acts throughout the life of the heating mask, distributing its active agents continuously.

## ■ SAFETY

The HeatMask+ is equipped with a heat protector to self-protect itself when it is used in a wrong way (use without any face inside, doubled up).

The HeatMask+, high frequency welded, is watertight: it can be cleaned simply with water and soap (do not use cleansing agents).

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## Technical data

2 sections:

- small section for the upper part of face
- large section for lower part of face, including chin and neck

Temperature setting in the large section

Treatment: hospital care quality

Twin colour: white/white & chocolate/toffee

Voltage: 230V or 115V

Standards:

EN60335-1 & EN60335-2-17

Warranty: 1 year